



Soft Nasal Cannula (adult)
SINGLE SIDE USAGE
Length - 7' (2.1 m)



UPODS LLC


2300 Lakeview Parkway, Ste 730
Alpharetta, GA 30009, USA

Assembled in USA

UNIT / Case - 100

LOT: see pouch

REF: PS-1007

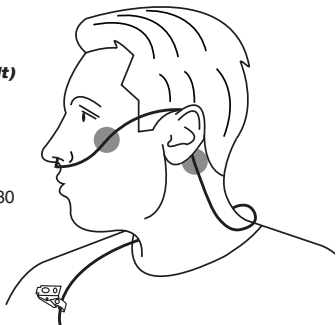

SUGGESTED PROCEDURE: *please see back panels*

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used with applicable prescription.
3. Drape tubing around back of neck, and over ear.
4. Insert nasal prong deeply yet comfortably into selected nostril.
5. Hold nasal prong in place in nostril, while molding the tube firmly behind and around ear.
6. Then press the tubing firmly over the tip of the cheekbone and into the contours of bony structure of face.
7. Adjust the tubing to rest lightly across back of neck to shoulder opposite the nasal insert. Turn head right and left. Apply clip to chest area clothing for secure placement.
8. Apply adhesive patches as shown in above diagram as needed.
9. Attach the device to oxygen source, ensure the connections are secure and start gas flow.
10. Confirm proper function prior to use.


CAUTION: Use of sterilization or cleaning solutions may leave this device nonfunctional

Uni-flo₂ is a trademark of UPODS, LLC of Alpharetta, GA • Copyright 2016 by UPODS, LLC • Patent Pending




Rx Only


1

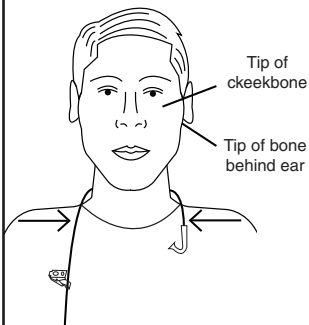

2

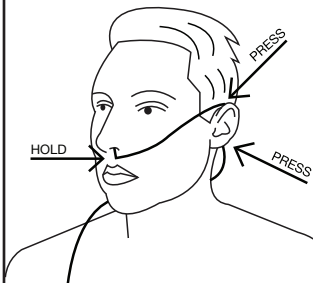

3

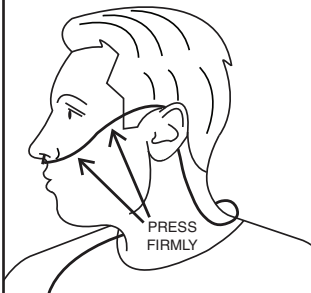

4

